

Sperm
360°



www.spermprocessor.com



Sperm Processor Pvt. Ltd.

6, Welcome Nagar, Garkheda,
Aurangabad (MS) - 431005, India.

Ph.: +91 240 6603800

Fax: +91 240 2341694

Email: info@spermprocessor.com

Morphology-D

evaluation of sperm morphology
by Diff-Quik Staining method

REF SP/SFT/MP-005-A

IVD



User Manual





Turnaround time for test: 1min



Store at: 2°C - 8°C after receiving

Index

Concept	01
Specimen Preparation	02
Special Instructions	02
Kit Contents	03
Equipments	05
Disposable Materials	05
Procedure	06
Morphology of Normal Sperm	10
Examination	12
Result Interpretation	15
Coverslipping Stained Slides	18
Safety & Environment Precautions	19

Morphology-D

CONCEPT

Sperms in human ejaculate exhibit considerable **morphological variations** in size & shape of **Head & Acrosome**. Also changes are seen in degree of **nuclear vacuolations**, size of persisting **cytoplasmic droplets**, **mid-piece** disturbances & **tail** abnormalities.

Sperm is considered **normal** when **Head, Mid-piece, Neck & principal piece (Tail)** are normal.

Normal **head** has **smooth oval** configuration with well **defined** acrosome comprising of **40 - 70%** of head.

The **cytoplasmic droplet**, if present, should **not** be **more** than **one half** the **size** of sperm head.

The **mid-piece** should be slender, less than **1µm** in **width**, about **one & a half times** the length of the **head**, & attached **axially** to the head.

The **principle piece (tail)** should be straight, uniform, thinner than mid-piece, uncoiled & approximately **45 µm** long.

In order to achieve **accurate** sperm morphological **evaluation**, following **principles & steps** must be followed:

- **Correct preparation** of slides & smears.
- Correct **fixation**
- Correct **staining** of the smear
- Appropriate **magnification** used
- Number of **fields used** to evaluate the sperm & number of sperm evaluated.

- Semen sample is collected with :
 - **Abstinence period** of **2-7days**.
 - **Ideal collection** through **masturbation** in sterile container.
 - **Non-spermicidal polyurethane semen collection pouch (Sperm Collect™)** can be used when required.
- Semen sample is allowed to liquefy and then well mixed for performing test.

Special Instructions :


- **Hyperviscous** semen sample should be processed to bring towards normal viscosity. (**Viscosity-CH™** or **Viscosity-BR™** kit can be used)
- Severe **oligospermic** semen sample (i.e. sample with Sperm Concentration less than 5millions/mL) should be processed to concentrate the sperm concentration to around 8-10 millions/mL before performing the test.
- **Frozen semen** must be thawed at 37°C (with Sperm Warmer™) before performing test.

- Fixative Solution (F1) : 100 mL
- Stain - I Solution (S1) : 100 mL
- Stain - II Solution (S2) : 100 mL
- Washing Medium (WM) : 60 mL

Other Reagents : Required But Not Provided In Kit :

- Distilled Water
- Xylene (Neoclear)
- Mounting Solution
- Immersion Oil

Kit Content Layout Diagram :

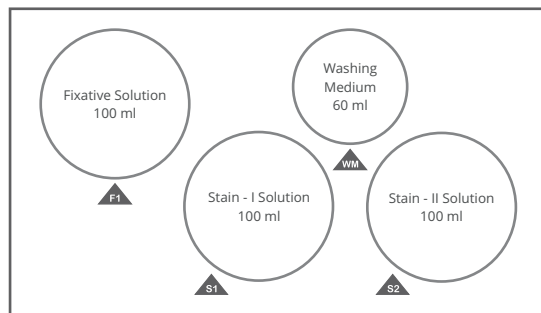

Storage Conditions :

- The kit should be stored in dark at 2°C - 8°C after receiving.
- Bring all the reagents to room temperature before use.
- Once opened, store reagents in the fridge protected from light.
- Expiry date is printed on the out side of the box.

REQUIRED BUT NOT PROVIDED IN KIT

- Microscope
- Controlled Temperature 37°C Dry bath (Sperm Warmer™ / Water bath)
- Set Of Pipettes
- Centrifuge Machine (Androspin™)
- Slide Warmer™
- Stopwatch
- Semen Analysis Chamber (Sperm Meter™)
- Microtip Box
- Staining Tray
- Glass Slide Stand
- Glass Slide Tray
- Coplin Jar

REQUIRED BUT NOT PROVIDED IN KIT

- Hand gloves
- Semen Collection Container
- Non-spermicidal Semen Collection Pouch (Sperm Collect™)
- Microtips
- Pasteur Pipettes
- Test Tubes
- Glass Slides
- Coverslips
- Filter Papers

A. Labeling :

Label plastic ware & disposable materials with appropriate **Patient ID** & **Sample ID**.

B. Semen Smear Preparation :

- **Evaluate** the **sperm concentration** of given semen sample and note it down.

• **Neat Semen Smear**

- If **Sperm Concentration is greater than 40 - 60 millions/mL**, place **5 μL** of liquefied semen sample on glass slide. Form the smear.



- If Sperm Concentration is **less than 40 millions/mL**, place **10 μL** of liquefied semen sample on glass slide. Form the smear.

**C. Processed Semen Smear :**

- Take **100 μL** of liquefied semen.
- Add **400 μL** of washing medium.
- Mix well & centrifuge at **2000 rpm** for **2 - 3** minutes.
- Discard supernatant.

- Adjust the sperm concentration between **40 - 60** millions/mL with washing medium & use **5 μL** to prepare the smear.



- If adjusted concentration is **less than 40** millions/mL, use **10 μL** to prepare the smear.



D. Test Procedure :

Step 1 : Allow smear to air dry (use Slide Warmer™)

Step 2 : Lay smear horizontally & cover the entire smear with **1 mL of Fixative** solution. Keep it for **15** seconds.



Step 3 : Drain off the **fixative** solution & allow smear to air-dry .



Step 4 : Lay smear horizontally & cover the entire smear with **1 mL of Stain - I** solution. Keep it for **12** seconds.

Step 5 : Drain off **Stain - I** solution

Step 6 : Lay smear horizontally & cover the entire smear with **1 mL of Stain - II** solution. Keep it for **8** seconds.



Step 7 : Drain off **Stain - II** solution

Step 8 : Rinse smear in **DW**

Step 9 : Drain off **DW** & clean the back of the slide with a filter paper.



Step 10 : Allow smear to air-dry (Use Slide-warmer).



Quick Glance



Examine
under **100x** (objective lens)

6 Morphology of Normal Sperm

- **Head** : Smooth oval configuration
- **Dimensions** : Length :- 5.0µm - 6.0µm
Width :- 2.5µm - 3.5µm
- **Acrosome** : Acrosome should comprise of 40 - 70% of the anterior sperm head.
- **Borderline Forms** : Consider all borderline forms as abnormal.
- **Neck / mid-piece** : No abaxial implantations - Slender approximately 1 µm in width & 6 - 7 µm in length. Cytoplasmic droplets >30% of head size are considered abnormal.
- **Tail** : Uniform, slightly thinner than the mid-piece, uncoiled with principal piece 45-50µm & terminal segment of 4-6µm.

Normal Sperm :

- If sperm dimensions for Head, Neck & Tail are within normal limits, then it is called as '**Normal Sperm**'.

Abnormal Sperm :

- When any one of the parameters of sperm for Head, Neck or Tail is not within normal limits, then it is called as '**Abnormal Sperm**' (**Strict Criteria**).

STRICT CRITERIA

Abnormal Sperm : Defect Details

- **Head :**

Size	Micro
	Macro
Shape	Tapered
	Pyriform
	Round
	Amorphous
Vacuole	Present
Acrosome	Absence of acrosome
	Small sized
	Large sized
- **Neck :**

Insertion	Bent insertions
	Asymmetrical
Size	Thin neck
	Thick neck
- **Tail :**

Insertion	Bent
Length size	Short
	Coiled
No. of tails	More than one

- Put a drop of immersion oil on dry smear.
- Examine the smear under microscope with the help of **100x** lens.
- Examine at least **200** sperm and evaluate followings as per **Kruger's strict criteria** / WHO criteria :
 - Normal Sperm
 - Abnormal Sperm
- In abnormal sperm, detailed abnormality (defects) pertaining to head, mid piece and tail should be noted down.

Calculations

- Calculate the percentage of normal & abnormal sperm.
- Calculate total number of head, mid-piece & tail defects of the abnormal sperm.

Normal reference value / range :

For Normal Sperm **4%** (3% - 4%)

- Calculate sperm deformity index and teratozoospermic index.

Sperm Deformity Index =


Total No. Of Defects / No. Of Sperm Evaluated

Teratozoospermic Index =

Total No. Of Defects / Total No. Of Abnormal Sperms

(As per fifth edition of WHO laboratory manual for examination and processing of human semen).


Schematic Representation :

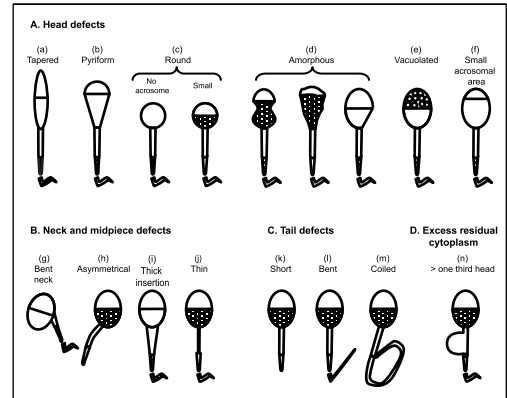

Adapted from Kruger et al., 1993 and reproduced by permission of MQ Medical.


(Fifth edition of WHO laboratory manual for examination and processing of Human Semen).

Reference Image :

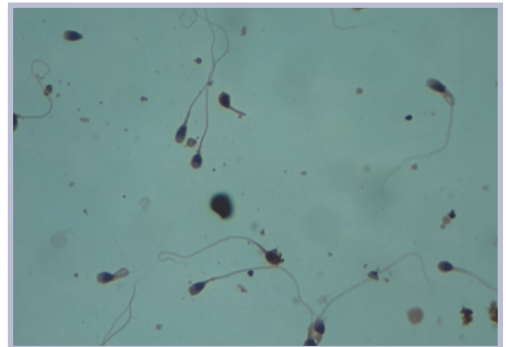



- Result interpretation is supported with -

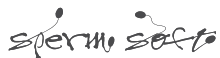

CASA with **Auto** &
innovative **Expert Mode**

- Individual test module - **Sperm Soft : Morph / Vitality** is also available.

Sperm Evaluation

- No. of Sperm Evaluated : _____
- Normal Sperm : _____
- Abnormal Sperm : _____

Total Sperm Defects

- Head Defects : _____
- Neck & Mid-piece Defects : _____
- Principal Piece (Tail) Defects : _____

Sperm Deformity Index : _____

Teratozoospermic Index : _____

Reference Value For Normal Sperm :

Normal : > 4%

Equivocal : $\geq 3\%$ & $\leq 4\%$

Abnormal : < 3%

Limitations :

- This test provides presumptive quantitative information of sperm Morphology.
- This parameter should be analyzed by a specialist.
- The result should be evaluated taking into account all clinical & laboratory findings related to the same sample.

Head Defects

- **Head Size**

Large Heads (LH) : _____

Small Head (SH) : _____

Normal Head (NH) : _____

- **Head Shape**

Tapered Heads (TP) : _____

Elongated Heads : _____

Amorphous Heads (AH) : _____

Round Head : _____

Normal Head (NH) : _____

- **Acrosomal Area**

Small Acrosomal Area : _____

Large Acrosomal Area : _____

Acrosomal Area Absent : _____

Normal Acrosomal Area : _____

- **Vacuoles**

Vacuolated Head (VH) : _____

- **Duplicate Head (DH)**

Duplicate Head (DH) : _____

Neck & Mid-piece Defects

- **Insertion**

Bent : _____

Asymmetrical : _____

Normal : _____

- **Size**

Thickened : _____

Thin : _____

Normal : _____

Principal Piece (Tail) Defects

- **Insertion**

Bent : _____

Normal : _____

- **Size**

Bent : _____

Short : _____

Coiled : _____

Irregular Tails : _____

Normal : _____

- **Principal Piece (Tail Defects)**

Loose Heads (Only Tails) : _____

(As per **fifth edition** of **WHO** laboratory manual for examination and processing of **human semen**).

Permanent Stained Slide :

- Dip dried stained - slide into Xylene (Neoclear) solution just prior to coverslipping.
- Place the mounting media on the slide.
- Place the coverslip on to the slide as quickly as possible to avoid air-drying & air bubbles.

- All patient samples & reagents should be treated as potentially infectious & the user must wear protective gloves, eye protection & laboratory coats when performing the test.
- The kit should be discarded in a proper biohazard container after testing.
- Do not eat, drink or smoke in the area where specimens & kit reagents are handled.
- Do not use beyond the expiration date which appears on the package label.
- It is recommended to use of gloves & face mask.

- Do not release the products used into the environment. Follow centre guidelines for the storage & disposal of toxic substances.
- Biological samples must be handled as potentially infectious.

Description of Symbols



consult instructions of use



product reference



lot number



use by



manufacturer



health surveillance device
for in-vitro diagnostic



contains sufficient for 'n' tests



temperature limitation



keep dry



CE mark (Conformité Européene)

Accreditations & Registered Certificates

- **ISO 13485 : 2003** Certified
- **CE** Certified
- **GMDN** Registered
- **US FDA** Registered

For more information & procedure videos



<http://www.spermprocessor.com/sft-morphology.html>



www.youtube.com/watch?v=YFLSHZE31N8

all logos are trademarks
and sole proprietary of Sperm Processor Pvt. Ltd.